

GEAR VIEW QC

- ✓ “It just works” – Solves day-to-day PACS administration issues
- ✓ Easy – View, import, edit, burn and send DICOM studies to PACS/EMR
- ✓ Fast – Visual editing to split/combine studies or add/remove images

A simple and powerful QC tool for PACS/EMR


PACSGear Gear View QC from Hyland Healthcare is a simple and powerful quality control tool that makes day-to-day workflow easier for picture archiving and communication system (PACS) administrators and technologists. Based on Gear View, PACSGear’s medical image and results visualization technology, Gear View QC provides intuitive functionality for viewing, importing, editing, burning and sending DICOM studies to PACS/ electronic medical record (EMR). Gear View QC lets PACS administrators and technologists fix common demographic errors in any DICOM field, as well as visually edit studies to split/combine exams or add/remove images. Studies can also be anonymized by masking burned-in patient information.

GEAR VIEW QC FEATURES

- Edit any DICOM patient, study, series or image tag.
- Visually edit studies directly by splitting/combining or adding/removing images.
- Anonymize DICOM studies by masking burned-in patient information.
- Import, burn and send DICOM studies to PACS/EMR.
- Interface to radiological information system (RIS) worklist or DICOM archive to provide automatic loading of patient demographics for ease of use.
- DICOM Modality Worklist (MWL) ensures an accurate match with patient studies.
- Restrict auto-adding of incoming studies into designated folders to specific days and times.
- Create aliases to send or forward studies to multiple destinations at once.

The Hyland logo is positioned in the bottom left corner of the page. It consists of the word "Hyland" in a white, serif font, set against a square background with a vertical gradient from green at the top to blue at the bottom.

Hyland[®]

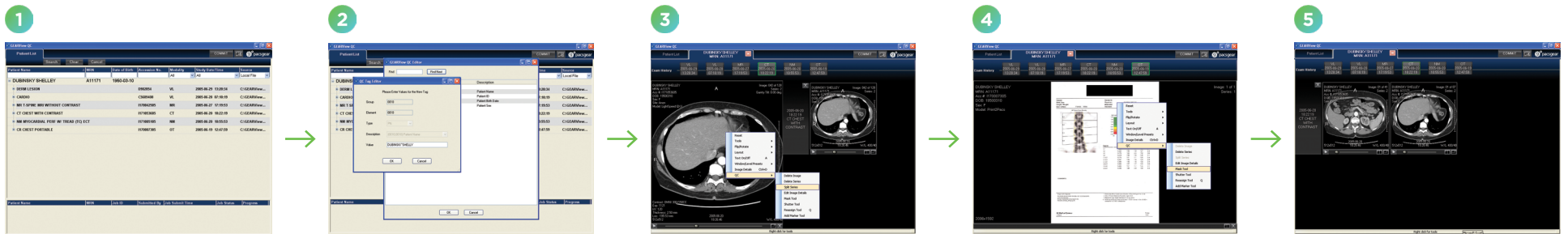

HOW GEAR VIEW QC WORKS

- 1 **Select patient:** Receive studies via DICOM Q/R, WADO, DICOM CD/DVD import, directly from modalities or via network transfers from outside facilities.
- 2 Edit any field.
- 3 Visually split into multiple series.
- 4 Remove burned-in patient information with mask tools.
- 5 **Send:** Send to PACS/EMR, burn DICOM CDs/DVDs with Media Writer, or export to local file.

Learn more at HylandHealthcare.com



EU Authorized Representative
EC REP Emergo Europe
 Molenstraat 15
 2513 BH The Hague
 The Netherlands
 LX-PMK-GVQC-BRO-EN-REVA



Hyland LLC
 4309 Hacienda Drive, Suite 500
 Pleasanton, California 94588 USA

Hyland®